

ORACollect[®]-RNA REF ORE-100

For IVD, Rx and For Use Under Emergency Use Authorization CE



DNAGENOTEK[™]
www.dnagenotek.com

4 DNA Genotek Inc.
3000 - 390 Palladium Drive
Ottawa, ON, Canada K2V 1G2
Subsidiary of QuidSense Technologies, Inc.

Contents: Contains 1 collection kit.
Intended use: For the collection and stabilization
of RNA from human saliva samples.



Warning and precautions: Ensure the sponge tip does NOT come into contact with any surface prior to collection. Choking hazard. Wash with water if the liquid comes in contact with eyes or skin. Do NOT ingest. See MSDS at www.dnagenotek.com

Emergo Europe, Prinsesvacht 20, 2554 AP The Hague, The Netherlands

Australian Sponsor:
Emergo Australia, Level 20, Tower II, Darling Park,
201 Sussex Street, Sydney, NSW 2000 Australia

Storage: 15°C / 25°C
Patent: (www.dnagenotek.com/legal/notices)
PD-LB-00563 Issue 4/2009-10

Label legend:

Rx For Prescription Use Only

Catalog number
 Lot number


CE Marking
 Manufacturer

Authorized Representative
 Use by

In vitro diagnostic medical device
15°C / 25°C Storage Instructions




1 検査キットを口の下あごの
歯茎とほっぺの間に、
できるだけ歯に触れないよ
うにして、奥まで差し込ん
で、左10回、下あごに擦
り付けるように唾液をを含
ませる。

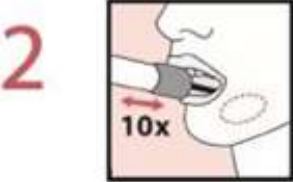

2 右側も同じように、
右10回、回しながら唾液
を含ませる



3 検査キットを上に向けて、
綿棒を回しながら外す。
この際に、中に入っている
液体をこぼさないようにお
願いします。

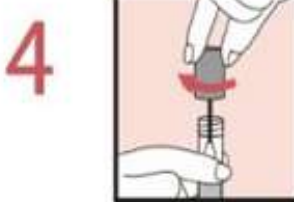

4 綿棒をキャップの中に差し
込み回しいれる。蓋をしっ
かりと閉める。



5 その後、絵のように
15回ほどしっかりと上下に
振ってください。

6 QRコードのついたステッカーを容器に貼る。

7 最後は赤いマークの付いた袋に入れる。