機体の操作方法 / How to Operate WHILL

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You can also check the operation method in the video

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Speed down

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Power

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Speed up

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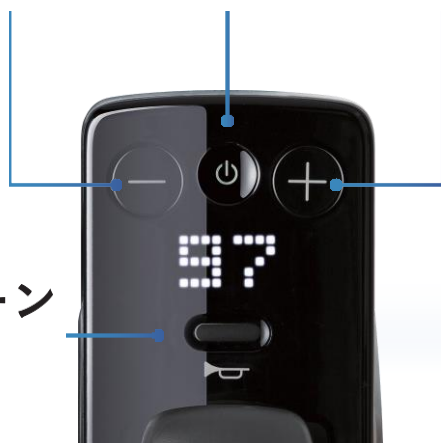
Turn left and right

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Reverse

ホーン
Horn

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Forward

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The device automatically stops when you release the control.

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- 1.Power OFF when getting on/off
- 2.Press power button after seated

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The speed is displayed first (1-4 levels).
Always start with "1", and only increase
the speed after learning to use the device.

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Next, the remaining battery level (%) will
be displayed. Please return the device
when it is low on battery.

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If the battery runs out or if you are worried about operating the device in a narrow space, **manually lower the red stoppers, and push it back by hand.**

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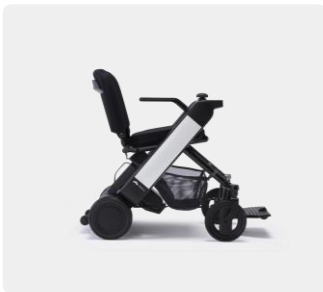
When leaving the device, **please lock it with the smart key***. * If a smart key is not provided, please turn off the device by pressing the power button.

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Please make sure to return the device by the end of the rental period.

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Please check the QR code for the insurance policy number, call 0120-119-110 and press "3" to report the accident. Automatic voice messages are in Japanese, but interpretation service will be available once you press "3". Once you reach an operator, please inform the insurance policy number.

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