

JOYcart

Simple Operation Manual


Please read carefully
before you start to use



Please see the
QR code for a
video of the
unfolding and
folding process.

■ How to unfold

1. Stand on the handle

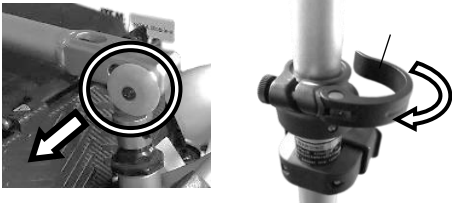


Raise the handle while pulling on the handle fixing knob. Loosen the height adjustment lever and adjust the height.

2. Straighten up the car body

Extend the vehicle forward, flatten the floor panels, and open them.



3. Move up a chair

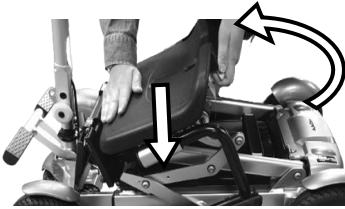

Pull the yellow lever backward while raising the chair forward.

If it is hard, press down firmly on the front of the chair.

4. Pull out the backrest.

Grip the fixture at the front of the chair and rotate the backrest to release the backrest and place it on the hook / Raise the backrest section.



5. Pullout armrest

Grasp the base of the armrest pole lightly and rotate upward while pulling backward.



Please see the QR code for a video of the unfolding and folding process.

■ How to drive

1. Check clutch lever position



Make sure clutch lever is in the lower position (drive mode). If it is on the upper side, be sure to turn it to the lower side.

2. Check battery pack




Check battery pack is set correctly.
In particular, push the handle in.

3. Insert a key



Please insert deeply and firmly.


4. Check the battery level

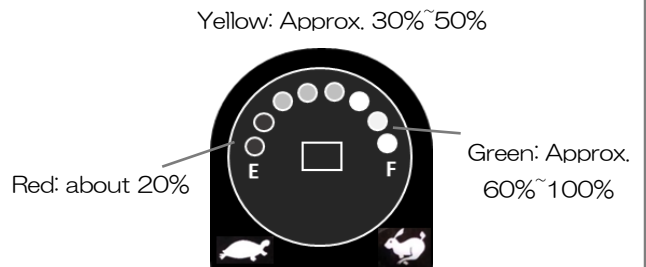

5. Forward and backward switch




Up Δ is forward /
Down ∇ is
backward

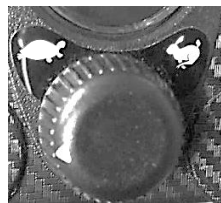

Turn the speed
adjustment dial on
the turtle side (slow),
gently squeeze the
accelerator lever.

6. Squeeze the accelerator lever

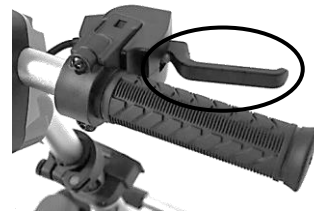

Gently gripping the accelerator lever to run.
Release the accelerator lever to automatic
braking.


Emergency braking when squeezed hard.

(caution*) 

■ How to fold


1. Put the backrest under the chair.

Remove the hook that hold the backrest in place and rotate the backrest to fit under the chair.

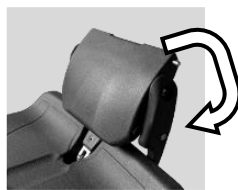

2. Pull out armrest

Lightly grasp the base of the armrest pole and rotate it downwards while pulling it back.

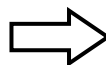

3. Straighten up the body

Fold the yellow part in the middle of the folding floor up.



4. Moving the chair down


Pull the yellow lever backwards and lower the chair



Press down firmly on the front of the chair.

5. stand on the handle


Loosen the height adjustment lever and tuck it down. Lower the handle down while pulling the handle fixing knob.

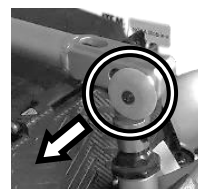

How to lift

Lift by the front step and carry handle.

* Do not lift with only the cowl, handle, and carrying part of the car body.

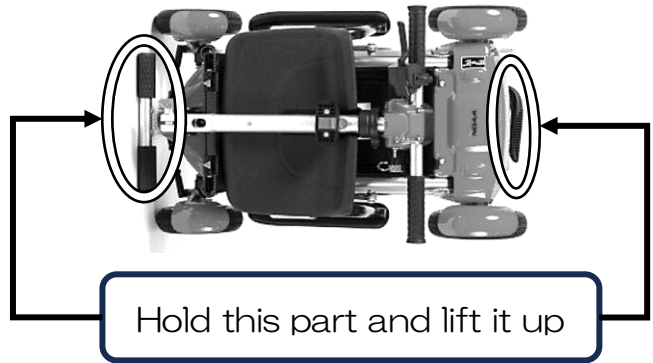

How to transport

You can trolley the vehicle sliding the yellow lever to the right, and extending the carry handle at the rear.



Battery

It can be easily removed by pulling up the handle.

Capacity: 292.32WH / stored at room temperature



Please recharge before use in winter or in cold regions.

How to Charge


Insert the connector into the charging port and plug the other end into a household outlet. When the pilot light turns from orange to green, charging is complete (about 6 hours to fully charged)




charger

When fully charged, be sure to unplug the cord. Do not use chargers from other companies. Do not store the battery in a hot or cold environment.

How to ride well

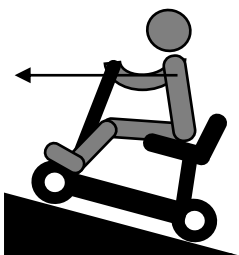

Forward the center of gravity

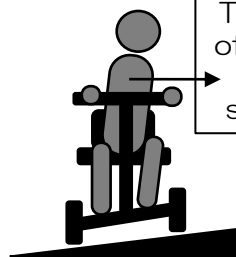

The center of gravity is on the slope side

Avoid slopes and slopes as much as possible.



If you have to drive, be careful and place your center of gravity on the side with a higher incline.

When driving on bumps, rickety roads, and curves, slow down and drive slowly.

Trouble shooting

Before you think it's a malfunction

Please check first before making an inquiry or requesting repairs.

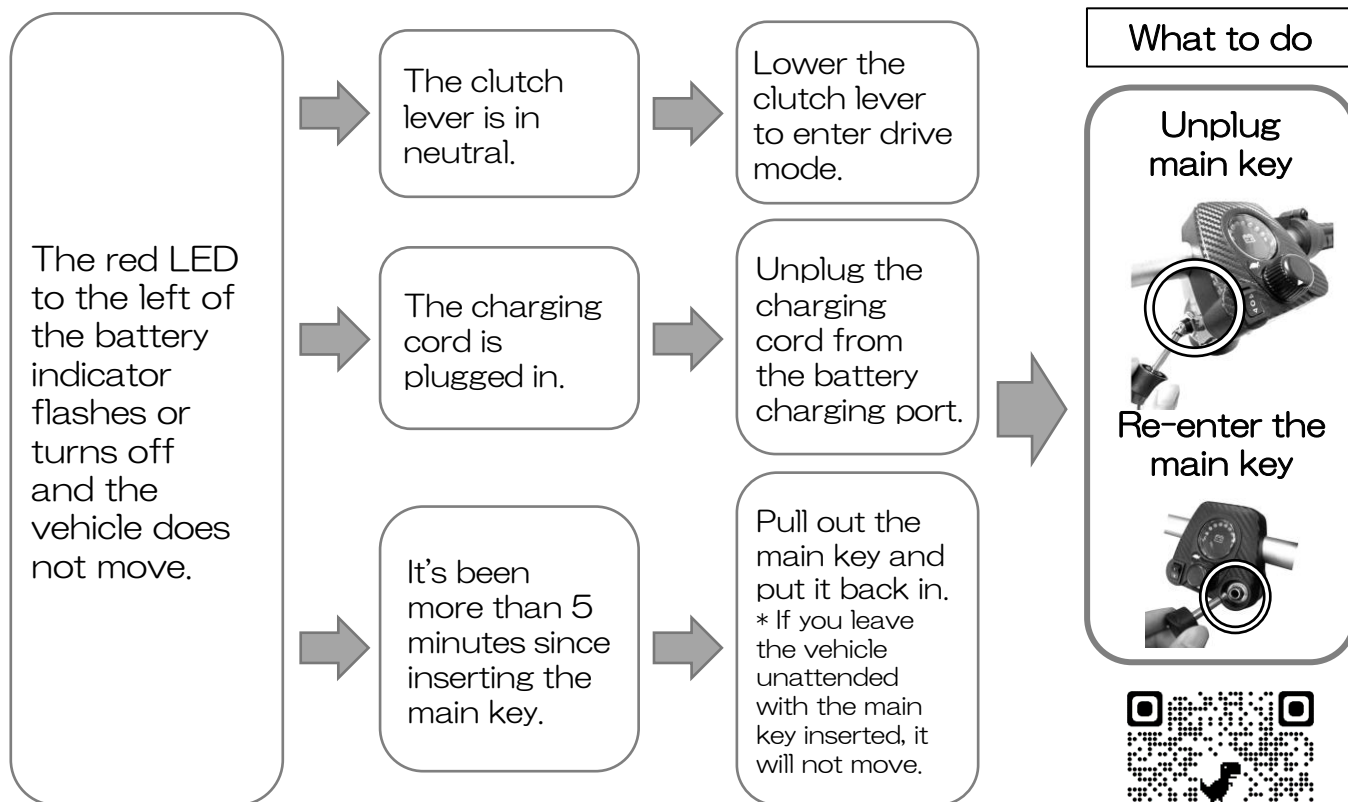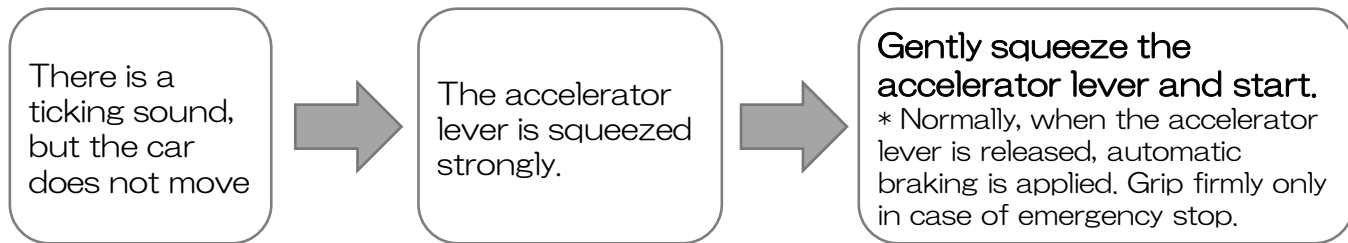

Other Announcements

This vehicle is classified as a wheelchair and is treated as a pedestrian. Drive on the sidewalk. When boarding a local bus, please fold it up and bring it with you as carry-on baggage. It is also possible to ride the bullet train or limited express train. Please contact the boarding station before using the train (JR, private railway, subway, etc.).

If you are flying by plane, please ask at the reception counter.

It can be loaded as luggage on the plane. Tell the airline that you will call the electric cart "NOAA Mobile" JAL Priority Guest Center 0120-747-707 (9:00~17:00) ANA Sky Assist 0120-029-377 (9:00~17:00) For other inquiries, please contact your airline

If you do not see any improvement after implementing each item, please contact the following
NOAA Co.,Ltd. info@noaa.jp